**TAVOLI CHIRURGICI AD USO VETERINARIO
VETERINARY SURGICAL TABLES**

**ERGOMED 99 - 2M-E
ERGOMED 99 - 3M-E**

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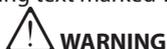
GENERAL INFORMATION

Thank you for choosing our product; we wish to remind you that it has been designed and produced paying the greatest attention to the safety of the operator, the efficiency of its work and to the protection of the environment.

In order to keep these characteristics in time, please read the instructions manual carefully and **comply meticulously with its content.**

Particular attention should be paid reading text marked with:

WARNING

it contains important safety instructions for use of the veterinary surgical table.

The Manufacturer is not to be held responsible for damages caused by:

- the failure to comply with what is contained in the instruction manual;
- a use of the veterinary surgical table different from those indicated in the **“INTENDED USE”** paragraph;
- a use failing to comply with the current standards concerning safety and accident prevention at work;
- tampering with the safety devices;
- incorrect installation;
- inadequate maintenance;
- modifications or operations that have not been previously authorised by the Manufacturer;
- the use of non original spare parts or parts that are not suitable for the veterinary surgical table model;
- repairs that are not carried out by a **SKILLED TECHNICIAN.**

DEFINITIONS

QUALIFIED TECHNICIAN: the Manufacturer or a person, specially trained and authorised by the Manufacturer to carry out supplementary maintenance operations and repairs on the veterinary surgical table. The operations on the electrical parts must be carried out by a **SKILLED TECHNICIAN** who is also a **QUALIFIED ELECTRICIAN**, meaning a person who is professionally certificated and trained to carry out the controls, installation and repair of electrical appliances, in a workmanlike manner and in accordance with the current standards of the country where the veterinary surgical table is installed.

USE AND RETENTION OF THE INSTRUCTION MANUAL

The instruction manual is an integral part of the veterinary surgical table and must be kept in a secure place, that allows their prompt consultation when required. It contains important instructions for the operator's safety and of those around him, as well as respect for the environment. In the event of his deterioration or loss, ask a copy to the Manufacturer. The instructions manual must accompany the veterinary surgical table in the event it changes owner.

The Manufacturer reserves the right to change, without notice, any changes required to the update and the correction of this publication.

SYMBOLS

The symbol:

WARNING

that marks certain parts of the text, indicates the high probability of personal injury if you do not follow their instructions and prescriptions.

The symbol:

CAUTION

that marks certain parts of the text, indicates the high probability to damage the veterinary surgical table if you do not follow their instructions and prescriptions.

EN

SPECIFICATIONS AND TECHNICAL DATA

The veterinary surgical tables **Ergomed 99**, in addition to a conventional veterinary surgery, permit to apply all techniques of positioning and skeletal traction, making them a real surgical instruments. They consist of a structure in stainless steel and interchangeable worktops, available in various versions (stainless steel, radiolucent Lexan[®], heated). The radiolucent worktops allow you to make the intraoperative fluoroscopy through them.

The **Ergomed 99 - 2M-E** standard version has up/down movement and longitudinal inclination (trendelenburg); **Ergomed 99 - 3M-E** version has in addition the movement of lateral tilting.

All movements are done with electric motors in low voltage (24V) and have a remarkable precision. Commands are given by sturdy pedal boards.

A guidance along the whole perimeter of the veterinary surgical table allows the housing of various optional accessories.

Both models are equipped with anti-tip stabilizers.

	Ergomed 99 - 2M-E	Ergomed 99 - 3M-E
ELECTRICAL CONNECTIONS		
Power supply	230 V 1~50 Hz	
Maximum absorbed power [W]	145	460
Maximum amperage [A]	0,63	2
Insulation class	I (protective earth)	
Maximum duty cycle [min]	1/19 (1 minute continuous use, followed by 19 minutes of rest)	2/18 (2 minutes continuous use, followed by 18 minutes of rest)
WORKTOP		
Maximum load [kg - lb]	150 - 330	
Operational area width [mm - in]	1030 - 40,5	
Operational area depth [mm - in]	400 - 15,7	
Minimum working height [mm - in]	770 - 30,3	800 - 31,5
Maximum working height [mm - in]	1070 - 42,1	1100 - 43,3
Maximum longitudinal tilting (trendelenburg)	+13°/-13°	
Maximum lateral tilting	-	+20°/-20°
Maximum moving speed [m/min - in/min]	0,6 - 23,6	
MOTORIZZAZIONE		
Number of motors	2	3
Motors supply	24 V dc	
NOISE LEVEL		
Sound pressure level [dB(A)]	less than 70	

MASS [kg] (*)	65	70
---------------	----	----

(*) Mass without accessories and without worktop.

Specifications and technical data are approximate.

The Manufacturer reserves the right to make all changes to the veterinary surgical table it deems appropriate.

EN

IDENTIFICATION OF THE COMPONENTS

Refer to Fig. 1, 2, 3, 4, 5 and 7 at the beginning of the instruction manual.

- | | |
|---|---|
| 1. Worktop | 22. Insertion reference for worktop |
| 2. Column | 23. Front foot maneuvering knob |
| 3. Base | 24. Rear feet |
| 4. Lateral stabilizers | 25. Lower door |
| 5. Front foot | 26. Cover |
| 6. Identification plate | 27. Universal clamp knob |
| 7. Warning plate | 28. Universal clamp lever |
| 8. Accessories guidance | 29. Bar seat |
| 9. Double pedal board | 30. Top door |
| 10. Single pedal board (Ergomed 99 - 3M-E only) | 31. Double pedal board connector |
| 11. Moving up button | 32. Single pedal board connector (Ergomed 99 - 3M-E only) |
| 12. Moving down button | 33. Locking connector lever |
| 13. Longitudinal tilting up button | 34. Supply cable connector |
| 14. Longitudinal tilting down button | 35. Main switch |
| 15. Lateral tilting clockwise (front view) button (Ergomed 99 - 3M-E only) | 36. Double pedal board bracket |
| 16. Lateral tilting counterclockwise (front view) button (Ergomed 99 - 3M-E only) | 37. Single pedal board bracket (Ergomed 99 - 3M-E only) |
| 17. Supply cable | 38. Double pedal board socket |
| 18. Double pedal board connecting cable | 39. Single pedal board socket (Ergomed 99 - 3M-E only) |
| 19. Single pedal board connecting cable (Ergomed 99 - 3M-E only) | 40. Supply cable socket |
| 20. Wheel | 41. Socket protective caps |
| 21. Wheel activation on/off knobs | 42. Pedal boards hooking holes |

IDENTIFICATION AND WARNING PLATES

WARNING

- If during the use one or more plates were deteriorating, contact the Manufacturer or a **SKILLED TECHNICIAN** for their replacement.

a) Identification plate.

The identification plate (6) shows the model of veterinary surgical table, the year of construction and the main technical data, including mass and maximum load. It is placed on the base (3).

b) Warning plate.

The warning plate (7) highlights some of the dangers that may arise in the use of veterinary surgical table. It is placed on the base (3). The meaning of the pictograms used is shown in the following table:

	Read the instruction manual		Not intended for the transport and/or lifting of persons
	Veterinary surgical table: not intended for persons		Do not perform any activities under the veterinary surgical table worktop if it is not mechanically locked

SAFETY DEVICES

Termofuse.

This device stops veterinary surgical table operation when overheating.

In this case, proceed as follows:

- remove the plug from the power socket;
- wait 10÷15 minutes, so as to cool the equipment;
- make sure the instructions for connection to the power supply are complied with (refer to the paragraph "**CHECKS AND CONNECTION TO THE POWER SUPPLY**");
- fit the plug back in the socket and repeat the starting procedures as described in the paragraph "**OPERATION**".

WARNING

- If the device is activated repeatedly, never use the veterinary surgical table without having to check it by a **SKILLED TECHNICIAN**.

STANDARD EQUIPMENT

Make sure the following are inside the pack of the product you have purchased:

- veterinary surgical table;
- worktop (type selected at the order);
- supply cable;
- double pedal board (up-down movements and trendelenburg);
- single pedal board (lateral tilting) (Ergomed 99 - 3M-E only);
- instruction manual.

If any problems arise, please contact the Manufacturer or a **SKILLED TECHNICIAN**.

OPTIONAL ACCESSORIES

WARNING

- Unsuitable optional accessories impair the operation of the veterinary surgical table and could make it dangerous. Use original optional accessories only, which have been recommended by the Manufacturer
- Respect all safety warnings of possible optional accessories that are used

It is possible to integrate the standard equipment of your veterinary surgical table with a wide range of accessories.

The following are those that can be directly installed on the veterinary surgical table, but there are many others of completion, for which please refer to the technical documentation, to the advertising

or to the website www.ad-maiora.eu.

VARIOUS WORKTOP

(One has to be chosen when ordering the veterinary surgical table).

The worktops are interchangeable and are available as follows:

- Lexan® radiolucent worktop;
- stainless steel worktop;
- heated worktop.

UNIVERSAL CLAMP

It is the fundamental element, which allows the connection of a large number of accessories to the veterinary surgical table. It is inserted into the guidance (8) and fixed in the desired position, tightening the knob (27). In the seats (29) can be entered, vertically or horizontally, the bars of a variety of accessories, allowing a great operating freedom; the bars are fixed in the desired position by tightening the lever (28) (refer to Fig. 7).

VETERINARY SURGICAL TABLE EXTENSION

It requires the use of an universal clamp.

It increases the surface of the veterinary surgical table worktop, keeping the overall size reduced. Indeed, with the use of the extensions to hold the head or limbs of the patient it is possible to place a large dog, while simultaneously allowing the surgeon to maintain a position close to the surgical theatre. This position is much more comfortable than the one you can take using veterinary surgical tables with large worktops, with which you need to keep your back arched forward during the procedure (refer to Fig. 8).

RETENTION BAND HOOK

It allows the anchorage and the tension of the animal retention bands. The anchor is done through the holes on the bands. The tension of the band can be adjusted by sliding the hook on the guidance (8). Once you reach the desired tightness, the hook is just released, and keeps the tightness by its self-locking mechanism (refer to Fig. 9).

HOLD-UP BAR FOR SURGICAL DRAPING

It requires the use of an universal clamp.

It holds up the surgical draping on the anaesthetist's side in order to allow easy access for monitoring purposes (refer Fig. 10).

ROD FOR DRIP

It requires the use of an universal clamp.

It allows to support the bottle for infusion of the patient without the need for a self-standing, reducing in this way the overall encumbrance (refer to Fig. 11).

ARMREST FOR MICROSURGICAL PROCEDURES

It requires the use of an universal clamp.

It has been specially designed to give stable support to the surgeon's arm during high-precision surgery. It can be autoclaved (refer to Fig. 12).

X-RAY CASSETTE TRAY

It requires the use of an universal clamp.

It is used for intraoperative radiography. By means of a bar kept overturned and supported by an universal clamp, the tray can be moved to the desired position underneath the patient, supporting the X-ray cassette during intraoperative radiography without direct operator's exposure (refer to Fig. 13).

SURGICAL TRAY

It requires the use of an universal clamp.

Overall 300 mmx400 mm (11,8 inx15,7in).

It allows all instruments to be within easy reach in the surgical theatre. If the surgical table is tilted, the tray can be rotated so that it is always in horizontal position (refer to Fig. 14).

SIDE POSITIONERS

They require the use of two universal clamps.

They make it possible to stabilize those patients who, because of their body conformation or position, tend not to be stable in the desired surgical position. The positioners are curved enough to provide adequate restraint without exerting dangerous pressure on the patient. One positioner is curved to the right and the other one is curved to the left (refer to Fig. 15).

TILTABLE FLUID COLLECTION PAN

It can be anchored at the end of the short side of the table using the clamp anchoring bars for fluid drainage. Being tiltable, it keeps the horizontal position also when the table is tilted. It can be connected to a discharge union in case a significant amount of fluid is drained (refer to Fig. 16).

TRACTION BAR

It allows traction to be performed by means of a mechanism of micrometric elongation, in a linear and progressive fashion. Once connected to the limb requiring traction using the most suitable technique, elongation is obtained by turning the handle at one of its ends. Since the mechanism is highly powerful, it is advisable to measure the applied force by means of a dynamometer (refer to Fig. 17).

STABILIZING ROD

It is used to prevent the decline of the plan when you push on the free end, especially during microsurgical procedures. This prevents the decline can change the focal point of the operating microscope, which would require an adjustment (refer to Fig. 18).

BIG DORSAL POSITIONER

It allows a stable dorsal recumbency of dogs bigger than 15 kg. It doesn't interfere with clamps' positioning in its area (refer to Fig. 6).

SMALL DORSAL POSITIONER

It allows a stable dorsal recumbency of cats and dogs til 15 kg. It doesn't interfere with the clamps' positioning in its area (refer to Fig. 6).

For further information, please contact the Manufacturer.

INTENDED USE

WARNING

- The veterinary surgical table is intended for the following use only:
 - use in veterinary clinics for diagnostic activity (including x-rays) and for surgical activity.
- The veterinary surgical table is not intended for:
 - diagnostic and surgical activity on persons;
 - lifting and/or transport of persons.
- The accessories (standard and optional) used with the veterinary surgical table must be of a type approved by the Manufacturer.
- The veterinary surgical table must not be used:
 - in environments in which explosive or corrosive atmospheres are present;
 - inside vehicles, ships or aircraft.

Any other use must be considered improper.

The Manufacturer cannot be considered responsible for possible damages caused by the improper or incorrect use.

UNPACKING

WARNING

- Wear goggles, safety shoes and protective gloves during unpacking operations to avoid injury to the hands, feet and eyes.
- The movement of the packed veterinary surgical table must be done through proper equipment, having a capacity of at least 250 kg (550 lb).
- The packing elements (pieces of strapping, pallets, cartons, plastic bags, paper clips, etc.) must not be left within reach of children as they are potential sources of danger.
- The disposal of packaging components must be performed in compliance with applicable regulations in the country where you installed the veterinary surgical table.
- Plastic packaging must not be discarded in the environment.
- The lifting of the veterinary surgical table must be made by an appropriate equipment, having a load capacity of at least 150 kg (330 lb).
- The manual lifting of the veterinary surgical table must be carried by two people.

The veterinary surgical table is delivered packed on a wooden pallet, covered with a cardboard and held together through two plastic straps.

To unpack, do the following:

- cut with a pair of scissors the two plastic straps;
- withdraw the cardboard, lifting it until you successfully extract;
- remove the wooden pallet: since the veterinary surgical table is heavy, this operation may be performed manually by two people, or by an appropriate lifting equipment, having a load capacity of at least 150 kg (330 lb). In both cases, advantage as gripping points areas S1 and S2 or S3 and S4 (refer to Fig. 1).

WARNING

- After having unpacked the veterinary surgical table, make sure it is intact. Also, check that the identification plate (6) and the warning plate (7) are present and legible. If in doubt, do not use the high-pressure cleaner and contact a **SKILLED TECHNICIAN**.

HANDLING AND TRANSPORTATION

WARNING

- Before handling or transporting the veterinary surgical table, carry out the operations indicated in the paragraph **“STOPPING”**.
- When transporting the veterinary surgical table, make sure it is well secured to the means of transport (with ropes or other suitable systems). Ensure to exclude the functionality of the wheels as shown below.
- Move the veterinary surgical table only on floors free of asperities and hollows.
- During handling pay attention to avoid damage of the power cord and of the pedal board cables.
- After the handling of the veterinary surgical table, provide to exclude the functionality of the wheels as shown below
- Do not handle the veterinary surgical table while there is still above the patient.

To handle the veterinary surgical table, proceed as follows:

- rotate fully counter-clockwise the knobs (21): this operation allows the contact with the floor of the wheels (20), so they can work;
- lift slightly (a few centimeters), the veterinary surgical table, exploiting the gripping point S1 (refer to Fig. 1) and move it up to the desired position;
- gently release the veterinary surgical table, allowing contact with the floor;
- rotate fully clockwise the knobs (21): in this way it is excluded the functionality of the wheels,

which no longer have any contact with the floor and the veterinary surgical table is in a stable position, thanks to the feet (24).

PRELIMINARY ACTIVITIES

EN

WARNING

- Do not use the veterinary surgical table if:
 - the power supply cable is damaged;
 - it is overturned or has sustained substantial impacts.If so, have it checked by a **SKILLED TECHNICIAN**.
 - Follow the instructions given in the “**ROUTINE MAINTENANCE**” and “**SPECIAL MAINTENANCE**” paragraphs.
- a) Lift the lever (33) of the double pedal board socket (38) and remove the socket protective cap (41) (refer to Fig. 2).
 - b) Plug the connector (31) of the double pedal board (9) to the the corresponding socket (38), pressing axially and then lowering the locking lever (33) (refer to Fig. 2).
 - c) Lift the lever (33) of the single pedal board socket (39) and remove the socket protective cap (41) (refer to Fig. 2).
 - d) Plug the connector (32) of the single pedal board (10) to the the corresponding socket (39), pressing axially and then lowering the locking lever (33) (refer to Fig. 2).
 - e) Lift the lever (33) of the supply cable socket (40) and remove the socket protective cap (41) (refer to Fig. 3).
 - f) Plug the connector (34) of the supply cable (17) to the the corresponding socket (40), pressing axially and then lowering the locking lever (33) (refer to Fig. 3).
 - g) Insert the worktop (chosen when ordering) in the edge (22) of the veterinary surgical table top frame (refer to Fig. 1).
 - h) Insert any universal clamps in the guidance (8).
 - i) Insert any extensions in its universal clamps.
 - l) Install any other accessories needed for the planned activity.

CHECKS AND CONNECTION TO THE POWER SUPPLY

WARNING

- A **SKILLED TECHNICIAN** must carry our the following:
 - make the connection of the surgical veterinary table to the electrical supply in accordance with the contents of IEC 60364-1, with particular reference to the presence of an efficient grounding circuit;
 - check that the electrical connection to the power supply is equipped with an adequate fuse and with a differential circuit breaker that can shut off the power if the leakage current to the ground exceeds 30 mA for 30 ms;
 - check that the power supply system is in accordance with the data indicated on the identification plate (6): the voltage should not vary by more than $\pm 5\%$.
- If the power supply cable is too short, an extension cord can be used, making sure that it is no longer than 10 m (33 ft) and that the section of the wires is at least 1,5 mm².
- **WARNING.** If an extension cord is used, the plug and socket must be of watertight construction.
- **WARNING.** Inadequate extension cords can be dangerous.
- Do not use reducers or adapters between the plug and the power socket.

WARNING

- Before using the veterinary surgical table, read carefully this instruction manual.
 - The veterinary surgical table is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge.
 - Children should be supervised to ensure that they do not play with the veterinary surgical table.
 - Comply with the safety instructions contained in the instruction manuals of any optional accessories that are used.
 - Before using the veterinary surgical table make sure it is positioned in a level, stable and dry location, to avoid it being upset or falling.
 - Before activating the movements (lateral tilting and trendelenburg), ensure that no foreign objects are in the vicinity of the veterinary surgical table: in particular, verify that the power cables of any power tools (clippers, saws, drills, etc.) are not damaged during movements.
 - When placing accessories in the areas of the guidance (8) adjacent to the column (2), make sure that any projections will not strike the same column when setting the higher values for longitudinal or lateral tilting.
 - **During all movements (up-down, lateral tilting and trendelenburg) always keep your hands above the worktop surface.**
 - Before handling the veterinary surgical table, carry out the operations indicated in the paragraph **“STOPPING”**.
 - During use:
 - carry out the operations described in the paragraph **“STOPPING”** when the veterinary surgical table is not used, before leaving it unattended, even for a short time and before any intervention: **in these cases do not leave the veterinary surgical table with the plug inserted into the socket current.**
 - the load must not exceed the maximum value expected for the veterinary surgical table (refer to the paragraph **“SPECIFICATIONS AND TECHNICAL DATA”**);
 - if the termofuse is activated, always remove the plug from the power socket and follow the instructions in the paragraph **“SAFETY DEVICES”**.
 - Pay particular attention to the instructions in the paragraph **“CHECKS AND CONNECTION TO THE POWER SUPPLY”**.
 - Do not remove the plug from the power socket by pulling the power supply cable.
 - Keep the power supply cable, possible extension leads, plugs and sockets dry. Never touch them with wet hands.
 - If the supply cord is damaged, it must be replaced by a **SKILLED TECHNICIAN**.
 - Do not remove the guards from the moving parts.
 - Do not carry out any maintenance on the veterinary surgical table if it is running.
 - Comply with the instructions in the paragraph **“INTENDED USE”**.
 - Do not modify in any way the supplied condition of the veterinary surgical table, in particular, do not modify the fixings, the connections and the guards.
 - Do not deactivate or tamper with the controls and safety devices.
 - If during use the identification plate (6) or the warning plate (7) were to deteriorate, contact a **SKILLED TECHNICIAN** to restore them.
 - It is forbidden to use water to extinguish a fire near the veterinary surgical table: use only extinguishing agents that do not create problems in the presence of electrical machines.
- a) Verify that the veterinary surgical table is in a stable position: in particular ensure that the knobs (21) are screwed in, so as the feet (24) instead of the wheels (20) are in contact with the floor.
 - b) Verify that the knob (23) of the front foot (5) is fully unscrewed, rotating it counter-clockwise.
 - c) If required by the activity that will be performed with the veterinary surgical table, activate the stabilizers (4): fully screw in knob (23), rotating it clockwise, then pull out one or both stabilizers

(refer to Fig. 1); once the stabilisers are positioned, fully unscrew knob (23), rotating it counter-clockwise.

- d) Arrange the pedal boards and their connecting cables in the best area to use them, without creating conditions of tripping and/or unintentional activation of their buttons.
- e) Check the main switch (35) is in "0" position; connect the plug into the socket; turn the main switch in "I" position.
- f) Verify the functionality of all movements, using the buttons:
 - (11) and (12), respectively to lifting or lowering the worktop vertically;
 - (13) and (14), respectively to longitudinally tilting upwards or downwards the worktop (trendelenburg movement);
 - (15) and (16), respectively to laterally tilting to the right or left the worktop (looking at the front of the veterinary surgical table) (**Ergomed 99 - 3M-E only**).

Refer to Fig. 2.
- g) Using the above mentioned buttons, put the worktop in the desired configuration.

CAUTION

- Do not use the veterinary surgical table:
 - if it is too noisy: in this case, have it checked by a **SKILLED TECHNICIAN**;
 - exposing it to direct sunlight, with ambient temperature above 40 °C (104 °F).

STOPPING

WARNING

- Always make sure that, once executed stopping procedures, no parts of the veterinary surgical table is moving.
 - After stopping the veterinary surgical table, pay attention to:
 - not leave it unattended if there are children, elderly and disabled people unsupervised;
 - arrange it in a stable position without danger of falls.
- a) Arrange the worktop approximately in an horizontal plane.
 - b) Turn the main switch (35) in "0" position.
 - c) Disconnect the veterinary surgical table, removing the plug from the socket.

CLEANING AND STORAGE

WARNING

- Every cleaning operation must only be carried out after executing the procedures described in the paragraph "**STOPPING**"; namely **with no moving parts and with the power supply disconnected**.
 - Any cleaning operation must be done with the veterinary surgical table positioned on a flat surface, in conditions of safe stability.
 - During cleaning, always wear adequate protective clothing.
 - For cleaning the veterinary surgical table do not use solvents or thinners.
- a) Carry out the operations indicated in the paragraph "**STOPPING**".
 - b) If present, drain the liquid from the tiltable fluid collection pan.
 - c) Clean and rinse surfaces that may come into contact with materials contaminated by body fluids. All surfaces (metal or plastic) should be cleaned with detergents and disinfectants containing no solvents or thinners. Avoid products that have explicit incompatibility with stainless steel or plastic. If the compatibility of the products in use is not confirmed, it is recommended to provide tests of contact on areas not intended for direct contact with the patient (for example the underside of the worktop).

We do not recommend the use of abrasive products.

When finished, the materials used for sanitizing must be sent for disposal or sterilization, in accordance with the rules of the territory where the veterinary surgical table is used.

During operations, or at the end of the same, you may want to impose on the worktop a longitudinal inclination as to facilitate the drainage of the products used for sanitizing. In any case, all operations must be conducted in the manner prescribed by the Veterinarian.

It is advisable to carry out the cleaning operations at the end of each use and in any case, according to the needs provided by the Veterinarian.

- d) Rewind and store with care the power supply cord and the connecting cables of the pedals boards.
e) Put the pedal boards in the special supports (36) and (37), using the holes (42) (refer to Fig. 4 and 5).

MAINTENANCE

WARNING

- Every maintenance operation must only be carried out after executing the procedures described in the paragraph “**STOPPING**”, namely **with no moving parts and with the power supply disconnected**.
- Any maintenance operation must be done with the veterinary surgical table positioned on a flat surface, in conditions of safe stability.
- **WARNING.** To ensure the veterinary surgical table safety, use only original spare parts provided by the Manufacturer or approved by the Manufacturer.
- Please also observe any laws, issued by local and/or national authorities of the country where the veterinary surgical table is used.

ROUTINE MAINTENANCE

For routine maintenance refer to the information in the following table.

MAINTENANCE SCHEDULE	ACTION
Every time is used.	<ul style="list-style-type: none"> • Check the power cable and the pedal board connection cables. If one or more parts are found to be damaged, do not absolutely use the veterinary surgical table and contact a SKILLED TECHNICIAN.

SPECIAL MAINTENANCE

WARNING

- Special maintenance must only be done by a **SKILLED TECHNICIAN**.
- Do not carry out any activities under the veterinary surgical table worktop, before have it mechanically blocked.

For special maintenance refer to the information in the following table.

MAINTENANCE SCHEDULE	ACTION
Every year.	<ul style="list-style-type: none"> • Check tightening screws. • Check the safety devices.

CAUTION

- The data given in the table are approximate. It might be necessary to carry out maintenance more frequently in the case of particularly heavy work.

DISMANTLING

WARNING

- Before dismantling the veterinary surgical table, ensure it cannot be used, for example by cutting the power supply cable and make safe parts that could be dangerous for children that may use the veterinary surgical table to play with.
- Only trained personnel are allowed to dismantle the veterinary surgical table in accordance with the current regulations in the country where it is installed.
- Electronic products not included in the selective sorting process are potentially dangerous for the environment and human health due to the presence of hazardous substances.

This product bears the selective sorting symbol for waste electrical and electronic equipment (WEEE - Directive 2012/19/UE):

Contact the manufacturer for end-of-life disposal.

TROUBLESHOOTING

WARNING

- Every operation must only be carried out after executing the procedures described in the paragraph **“STOPPING”**.
- In the event the correct function of the veterinary surgical table cannot be restored by means of the information in the **“TROUBLESHOOTING”** table, contact a **SKILLED TECHNICIAN**.

FAILURES	CAUSES	SOLUTIONS
Pressing one or more buttons of the pedal boards, no movements are activated: the main switch (35) is in “I” position, but it is not lighted.	A safety device of the system to which the veterinary surgical table is connected (fuse, differential switch, etc.) has tripped.	Reset the protection device. In case of a new triggering, do not use the veterinary surgical table and contact a SKILLED TECHNICIAN .
	The plug of the power supply cable is not inserted correctly.	Follow the instructions in the paragraph “SAFETY DEVICES” .
	The connector (34) of the main cable (17) is not fully inserted into the socket (40).	Disconnect the connector and reconnect it correctly.
Pressing one or more buttons of the pedal boards, no movements are activated: the main switch (35) is in “I” position and it is lighted.	The possible all poles switch to which is connected the veterinary surgical table is in the off position “0”.	Disconnect the power plug from the socket and reconnect it correctly. Move the all poles switch in the on position “I”.
	The thermal fuse is tripped.	Follow the instructions in the paragraph “SAFETY DEVICES” .
Pressing one or more buttons of the pedal boards, respectively of the pedal board (9) or (10) is not correctly inserted into the corresponding socket (37) or (38).	The connector (31) or (32), respectively of the pedal board (9) or (10) is not correctly inserted into the corresponding socket (37) or (38).	Disconnect the connector/s and reconnect it/them correctly.
	You have reached the limit of the movements.	Activate the movement in the opposite direction.
Pressing one or more buttons of the pedal boards, the corresponding movements stop during operation, while the remaining ones continue to operate normally.	One or both of the knobs (21) are partially unscrewed, so it is possible for the wheels (20) contact the floor.	Verify that the knobs (21) are screwed in so as to prevent the wheels (20) contact the floor.
	Ongoing activities require the use of the stabilizers (4).	Set the stabilizers, in accordance with paragraph “Operation”, according to the most appropriate configuration for the task.
The veterinary surgical table is not stable.	Excessive load.	Reduce the load.

SECTION ONLY FOR USE BY SKILLED TECHNICIAN

WARNING

- **This part of the manual is only for use by SKILLED TECHNICIAN and is not meant to be used by the end user.**
- Every repair operation must only be carried out after executing the procedures described in the paragraph **“STOPPING”**, namely **with no moving parts and with the power supply disconnected**.

- Do not carry out any activities under the veterinary surgical table worktop, before have it mechanically blocked.
- Any repair operation must be done with the veterinary surgical table positioned on a flat surface, in conditions of safe stability.
- **WARNING.** To ensure the veterinary surgical table safety, use only original spare parts provided by the Manufacturer or approved by the Manufacturer.
- Please also observe any laws, issued by local and/or national authorities of the country where the veterinary surgical table is used.

REPAIR - REPLACEMENT OF A PISTON

In case you must carry out a linear actuator replacement, provide as follows.

- a) Lift the worktop to the maximum allowed height.
- b) Place an adequate shimming in height and load, below the working plane, in order to lock it mechanically.
- c) Remove the doors (25) and (30), by loosening the screws.
- d) Remove the covers (26), by loosening the screws.
- e) Remove the seeger and the locking pins of the faulty linear actuator: without adequate mechanical shimming, this operation would result in the fall of the worktop.
- f) Remove the faulty linear actuator, replacing it with a new one of the same type.
- g) Reinstall covers (26) and doors (25) and (30).
- h) Remove the shimming supporting the worktop.
- i) Reconnect the power supply.
- l) Check the functionality of the new linear actuator.

EC/EU declaration of conformity

(according to Directives 2006/42/EC, 2014/30/EU, 2011/65/EU and its subsequent modifications)

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authorised to compile the technical file
declares under its own responsibility that the veterinary surgical table:

ERGOMED 99 - 2M-E

ERGOMED 99 - 3M-E

with S/N starting from S4026

is in accordance with the above-mentioned Directives.

The following Standards have been consulted for the conformity verification:

EN ISO 12100	EN 1570-1	EN 61340-5-1
EN 61340-5-2	EN 55011:2009	EN 55022:2010
EN 61000-3-2:2006	EN 61000-3-3:2008	EN 50581

Cavriago, 21/10/2016

Rovesti Gian Luca
(Legal Representative of AD MAIORA S.r.l.)

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Ad Maiora S.r.l.